

Subsalve Oxygen Treatment Hood Neck Seal Information

Neck seals are offered in either latex or silicone. To best match hood use with the individual, measure the individual's neck circumference, and then reference the below sizing guide.

Each neck seal has considerable stretch and can accommodate a wide range of neck circumferences out of the box. Seals can additionally be trimmed for added comfort.

We recommend first experimenting with various sizes to understand the stretch characteristics and comfortable range.

Material	Size	Neck circumference	Note
Latex	S	8.5" to 11"	
Latex	M	Up to 14"	
Latex	L	Up to 16"	
Latex	XL	Up to 17+"	
Latex	universal	11" to 16.5" without trimming	Can be trimmed for added comfort
Silicone	universal	11" to 16.5" without trimming	Can be trimmed for added comfort

Advice on Neck Seal Care

Both latex and silicone neck seals are durable though must be handled with care. Overstretching, sharp objects, and/or poor trimming can cause the seal to tear.

Do not stretch the seal more than is required to place it on an individual. Use two hands to stretch the seal from inside the opening. Two individuals (4 hands) is even better when placing on another individual. Once in place, run a finger along the inside of the seal to ensure it lays flat against the individual's neck for comfort.

Avoid sharp objects such as finger nails and rings.

Use a product such as talcum powder to minimize frictional strain on the seal and for personal comfort. Talcum can be used under the seal if it will be worn for a long period of time.

When trimming seals, use sharp scissors. Cut edges should be as smooth and straight as possible. Do not trim across trim lines. Avoid sharp edges and nicks.

